

# Jobskin® Classic

For other languages, please download from: [www.jobskin.co.uk/wash](http://www.jobskin.co.uk/wash)

## How to care for your new made-to-measure garment

Before washing your garment, please follow these simple rules to help protect and prolong the life of your garment:

### REMEMBER

- Wash your garment daily to rejuvenate the fabric
- Turn inside out and fasten zips and hook and loop tabs
- Wash dark colours separately
- Do not dry in direct sunlight, on a radiator or in a tumble dryer
- DO NOT USE A FABRIC CONDITIONER OR A DETERGENT THAT CONTAINS AN ADDED FABRIC CONDITIONER

### HAND WASH

- If hand washing, roll your garment inside a thick towel to remove any excess moisture
- Rinse well and dry flat on a clean, dry towel. Do not wring

### MACHINE WASH

- Use a 40 wash cycle
- Use a non-biological mild detergent, DO NOT use fabric conditioner in any format
- Use a laundry bag if available



Hand Wash



Machine Wash at 40°C



Do Not Bleach



Do Not Iron



Do Not Dry Clean



Do Not Tumble Dry



Do Not Wring



Dry Flat



Drip Dry

Please refer to separate insert if your garment contains Silon-TEX II